MEDIKRO

SpiroSafe-flowtransducere SpiroSafe Pneumotachograph SpiroSafe Flow Transducers Transductores de flujo SpiroSafe Voomuundurid SpiroSafe SpiroSafe virtausanturi Capteurs SpiroSafe

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EN SpiroSafe Flow Transducer operating and safety instructions

Medikro M9256 SpiroSafe disposable flow transducers employ the screen type pneumotach principle. Each SpiroSafe flow transducer is inspected and measured to ensure quality.

Warnings

Conditions or practices that may cause patient injury, illness, or death are identified in the following warning statements.

- WARNING** The disposable flow transducer is a single-use product only. **WARNING** Do not cleanse the flow transducer. This increases the risk of cross-contamination and significantly alters resulting measurements. In cases where a disposable SpiroSafe flow transducer is cleaned, Medikro cannot guarantee the accuracy of the spirometry system. **WARNING** Disposable flow transducers are packaged as non-sterile. **WARNING** Check that there is no foreign objects inside SpiroSafe flow transducers. **WARNING** Do not mix clean and used flow transducers. **WARNING** Do not mix new, clean, disposable flow transducer for each patient.

Cautionary statements

Conditions or practices that may cause equipment failure, equipment damage, or data loss are identified in the following cautionary statements.

- CAUTION** The disposable flow transducer M9256 should be used with the following spirometers: Medikro Nano, Medikro Primo, Medikro Pro or Medikro Duo. **CAUTION** Once opened, packages should be protected by covering. **CAUTION** Spirometers should be calibrated daily, in line with the directions provided in the user manual. **CAUTION** Never calibrate spirometers with a used flow transducer. This would entail a risk of cross-contamination via the calibration syringe. **CAUTION** Patients can easily hold disposable flow transducer with their teeth (no hands required). **CAUTION** Whenever a new SpiroSafe flow transducer package is opened, please check the calibration code on the label on the exterior of the package. If the calibration code differs from the one currently in use, new code must be employed before testing is resumed. Using a wrong code may result in inaccurate results.

Before using a new flow transducer package

When a new SpiroSafe flow transducer package is used, please check the calibration code on the label on the exterior of the package. If the calibration code differs from the calibration code currently in use, a new calibration code must be employed before testing is resumed. Please consult the directions provided in the user manual.

Instructions for disposal and materials

For GTIN-codes: 06420099000554, 06420099000561, 06420099000578, 06420099000585, 06420099000592, 06420099000608, 06420099000615

Materials: flow transducer: PEHD, internal bag: PELD, single-pack film: OPP+PELD and external carton: cardboard

Please dispose of the flow transducer, internal bag and external carton in accordance with national or local laws or your facility's disposal policy. All materials are recyclable or may be burned. A product safety sheet is available, upon request, for each material.

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