MEDIKRO # M9242

SpiroSafe-flowtransducer

SpiroSafe Pneumotachograph

SpiroSafe Flow Transducers

Transductores de flujo SpiroSafe

Voomuundurid SpiroSafe

SpiroSafe-virtausanturi

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SpiroSafe mondstuk

SpiroSafe flowtransduser

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Transduttore de Fluxo SpiroSafe

SpiroSafe-flödesgivare

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EN SpiroSafe Flow Transducer operating and safety instructions

Medikro M9242 SpiroSafe disposable flow transducers employ the screen type pneumotach principle. Each SpiroSafe flow transducer is inspected and measured to ensure quality.

Warnings

Conditions or practices that may cause patient injury, illness, or death are identified in the following warning statements.

WARNING! The disposable flow transducer is a single-use product only.
WARNING! Do not cleanse the flow transducer. This increases the risk of cross-contamination and significantly alters resulting measurements. In cases where a disposable SpiroSafe flow transducer is cleansed, Medikro cannot guarantee the accuracy of the spirometry system.
WARNING! Disposable flow transducers are packaged as non-sterile.
WARNING! Check that there is no foreign objects inside SpiroSafe flow transducers.
WARNING! Do not mix clean and used flow transducers.
WARNING! Use a new, clean, disposable flow transducer for each patient.
WARNING! Check that there are not defects or damage in the product package. Do not use in case package is damaged.

Cautionary statements

Conditions or practices that may cause equipment failure, equipment damage, or data loss are identified in the following cautionary statements.

CAUTION! The disposable flow transducer M9242 should be used with the following spirometers: Medikro SpiroStar and Medikro SpiroMaster.
CAUTION! Once opened, packages should be protected by covering.
CAUTION! Spirometers should be calibrated daily, in line with the directions provided in the user manual.
CAUTION! Never calibrate spirometers with a used flow transducer. This would entail a risk of cross-contamination via the calibration syringe.
CAUTION! Patients can easily hold disposable flow transducer with their teeth (no hands required).
CAUTION! Whenever a new SpiroSafe flow transducer package is opened, please check the calibration code on the label on the exterior of the package. If the calibration code differs from the one currently in use, new code must be employed before testing is resumed. Using a wrong code may result in inaccurate results.
CAUTION! Consult user manual (M8304, Medikro Spirometer, User Manual) for additional information.

Before using a new flow transducer package

When new package of SpiroSafe flow transducer is taken into use, check the production lot number from the label attached to the outer surface of the package. If this lot number differs from the lot number currently used, new linearisation file must be taken into use before resuming testing.

SpiroSafe flow transducer package include a CD, which contains the linearisation file data. The included CD is for use with the Medikro Spirometry Software only. Check the linearisation file of the SpiroSafe flow transducer and change it when needed in the Calibration window. Select the right linearisation file from the list box or select "New File" button. Follow the instructions given by the Medikro Spirometry Software. Or go to: Device -> Settings -> Spirometry -> Linearisation file
Select the right linearisation file from the list box. If the file is not listed, please select "New file" (at the top of the list) and follow the instructions given by the Medikro Spirometry Software.

Additional Instructions

Materials: flow transducer: PEHD, internal bag: PELD, single-pack film: OPP+PELD and external carton: cardboard
Please dispose of the flow transducer, internal bag and external carton in accordance with national or local laws or your facility’s disposal policy. All materials are recyclable or may be burned. A product safety sheet is available, upon request, for each material.
Storage and Transportation Environment: Temperatures between −20 °C (-4 °F) and +50 °C (122 °F)
Operation Environment: Temperatures between +10 °C (50 °F) and +40 °C (104 °F)

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