


Medikro Spirometry Software • Quick Instructions Guide

Flow Chart of Spirometry:

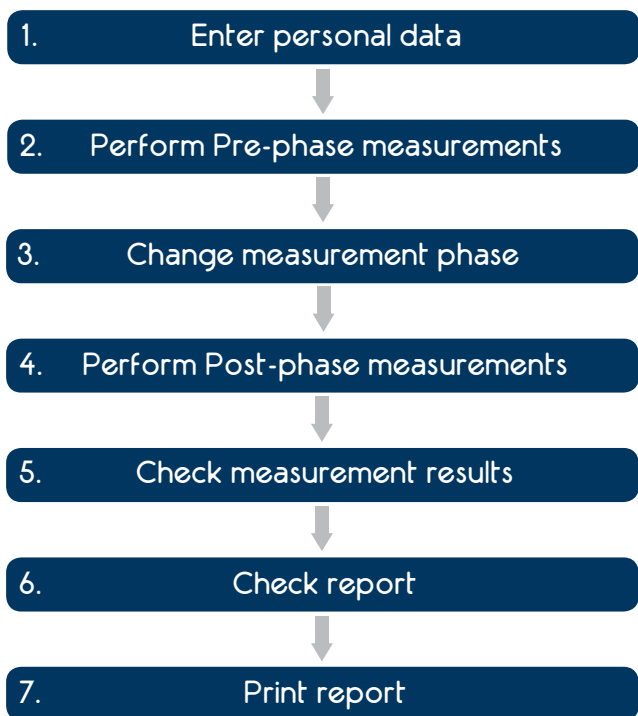

Other guidelines:

- ERS and ATS recommend that spirometers calibration is checked with 3000 ml calibration syringe daily. Calibration log file will store all calibration check values.
- Operator should make sure that the reference values correspond with the patient data.
- Operator should make sure that correct SpiroSafe calibration code is used.
- If you do not have Medikro Pro or Medikro Ambi measuring device you should verify daily the operating parameters: temperature, pressure and humidity.
- Operator should coach the patient before the session on the proper test manoeuvre.
- Operator should make sure that each patient would use a new and clean SpiroSafe flow transducer.

MEDIKRO
 Pioneerinkatu 3, FI-70800 Kuopio, Finland | P.O.Box 54, 70101, Kuopio
 +358 44 777 7007
 support@medikro.com | www.medikro.com

