


Made by: 2018-03-21 EK
 Checked by: 2018-03-21 EL
 Accepted by: 2018-03-21 MH

M8320
 1.1

Medikro Grip Operating and Cleaning Instructions

Operating Instructions

Push the flow transducer down onto the patient grip until it is secure.

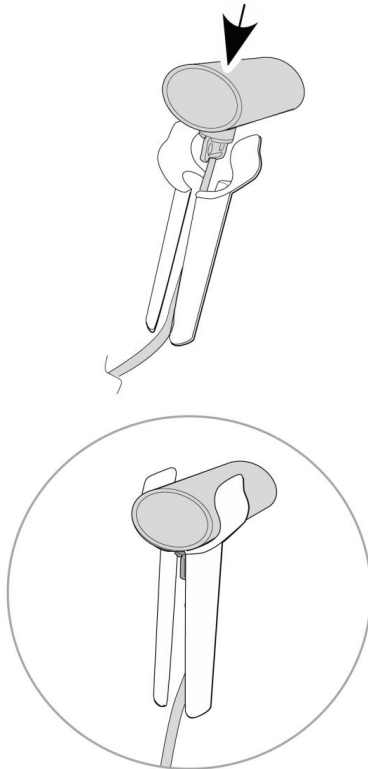

Warnings

A warning statement in this installation guide identifies a condition or practice, which could lead to patient injury, illness, or death.

WARNING Keep the reusable patient grip clean. Patient contact with contaminated equipment can spread infection.

WARNING The grips are packaged as non-sterile.

Cautions

A caution statement in this installation guide identifies a condition or practice, which could lead to equipment failure, equipment damage, or data loss.

CAUTION Use this grip with the following spirometers: Medikro Nano, Medikro Primo and Medikro Pro.

CAUTION Clean the patient grip after each patient use.

Cleaning Instructions

Clean on a routine basis according to your facility's protocols and standards or local regulations.

The following agents are compatible with the patient grip:

- 70 % isopropyl alcohol
- 10 % chlorine bleach solution.
- 55-60 % ethanol

Cautions

A caution statement in this installation guide identifies a condition or practice, which could lead to equipment failure, equipment damage, or data loss.

CAUTION When cleaning the patient grip, avoid using cloths or solutions that include quaternary ammonium compounds (ammonium chlorides) or glutaraldehyde-based disinfectants.

CAUTION Note disinfect according to your facility's protocols and standards or local regulations.

Disposal Instruction and materials

For GTIN-codes: 06420099000684 and 06420099000691

Materials: Grip: PA66, Single pack film: OPP+PE and External carton: Cardboard, External bag: PP

Dispose of the grip, the internal bag, and the external carton according to national or local laws or according to your facility's disposal policy. All materials are recyclable or burnable. The product safety sheet is available, upon request, for each of the materials.

The CE mark on this product indicates that it is in conformity with the provisions of Council Directive 93/42/EEC.

CAUTION Consult User Manual and other accompanying documents.

Medikro Oy

Mail address: P.O.Box 54, FI-70101 Kuopio, Finland
 Street address: Pioneerinkatu 3, FI-70800 Kuopio, Finland

Tel: +358 44 777 7003
 E-mail: sales@medikro.com
 Home page: www.medikro.com

Business Identity Code: 0288691-7
 V.A.T.: FI02886917
 Domicile: Kuopio